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Instructions for use URIVETTE® applies to the products URIVETTE® 10 ml and 8 ml

Intended use

The URIVETTE® is intended for the collection, preparation and subsequent storage of urine samples for laboratory analysis. Using the supplied aspiration tube, the URIVETTE® can be filled from a urine collection container (e.g. urine beaker, 24-hour urine collection vessel) or via the Luer cone on a bladder catheter connection. The samples can then be safely transported in the tubes, checked with test strips and centrifuged.

Diagnostic tests from the URIVETTE® are possible from urine samples from patients regardless of age and gender. Different measured values can be determined from the urine sample and important information about the patient's state of health can be obtained.

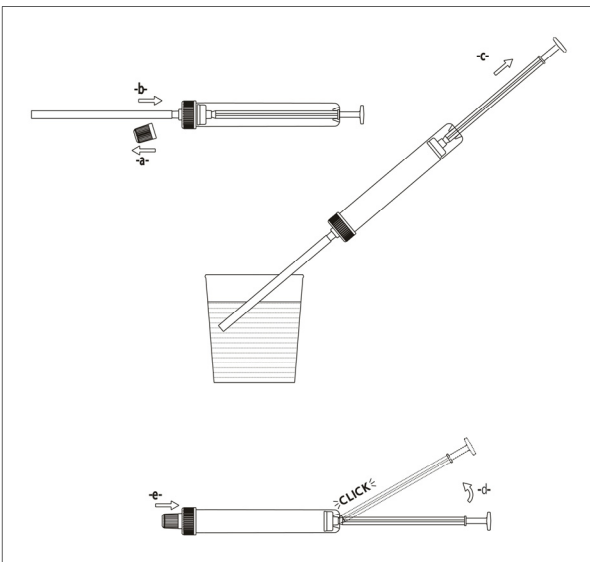
Features

The URIVETTE® is a CE-marked IVD product that is classified as class A. The sample container consists of an aspiration tube with graduation, a yellow screw plug with Luer cone and a cone cap. A suction tube is also included.

Notes

- Wear protective gloves and if necessary, other protective clothing!
- Use the product only within the specified shelf life.
- The storage time and temperature of a filled URIVETTE® depends on the stability of the parameters to be tested.
- Any violent handling of the product must be avoided.
- The product is only intended to collect urine.
- The product is for single use only!

Application

- 8) Remove the cone cap from the cone of the URIVETTE® and keep it ready for resealing (-a-).
- 9) Attach the suction tube to the cone (-b-).
- 10) Draw up the urine from the urine collection container by pulling back the piston. Make sure to remain above the container to avoid possible contamination (-c-).
- 11) NOTE: After filling the URIVETTE®, there may be residual urine in the suction tube. To avoid contaminating the user or the working environment, hold the URIVETTE® with the suction tube pointing downwards and remove it above a drain or the urine collection container and then dispose of it.
- 12) Then break off the piston rod (-d-).
- 13) Close the URIVETTE® by attaching the cone cap (-e-).
- 14) **Strip test:** open the URIVETTE® and carry out the strip test directly in the tube. The length of the URIVETTE® is adapted to the test strip.
- 15) **Sediment extraction:** Centrifuge URIVETTE® for sediment extraction (approx. 5 min at 400-800 g). During centrifugation, the sediment settles on the bottom of the flask.

Storage

Store in a dry place at 5°C to 25°C and protect from direct sunlight and humidity.

All serious incidents related to the product shall be reported to the manufacturer and the responsible authority of the member state where the user and/or patient is registered.

Symbol meaning

	Item number		Protect from sunlight
	Lot in YYX0000		Store in a dry place
	Can be used until YYYY-MM		IVD product
	CE marking		Do not use if the packaging is damaged
	Manufacturer		Follow the instructions for use
	For single use only		Temperature limits
			Unique device identifier

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KABE LABORTECHNIK GmbH